

Outdoor



Stannah

Frank will never forget the first time his grandfather took him to the allotment and gave him a taste of homegrown tomatoes. Now he has an Outdoor stairlift, he can check the tomatoes in his greenhouse, and follow his grandmother's recipe for the perfect tomato soup.



We are intensely proud to have stayed a family-owned independent business. It has allowed us to remain true to our commitment to making the highest quality stairlifts.

Our Outdoor stairlift for straight stairs is specially designed to work all year round. An ideal solution for either a flight of outside stairs or for just a few steps down to your garden.

The chair

The swivel seat allows you to get off the chair in complete safety and the joystick controls can be positioned on either arm, whichever is more comfortable. A protective cover is provided to keep your stairlift clean and dry when you're not using it.

The rail

Fitted to the stairs and not to the wall, the straight rail can only be fitted to solid concrete, solid wood or metal staircases.

New and reconditioned models

Our outdoor chairs are available to buy as either new or reconditioned models. All pre-used stairlifts are reconditioned and rigorously tested by our team of engineers using a 42 point check, before being fitted.

The service

The reliability of our stairlifts comes from the quality of the engineering and design. For your added peace of mind, all our new and reconditioned products are backed up by a full warranty and service package. Just like any piece of machinery, we recommend that your stairlift is serviced annually by a trained engineer. We will arrange with you to come and visit to give the stairlift a thorough and complete service, putting it through its paces.

You won't have to wait for service or help as our nationwide teams of engineers are locally-based. They are on standby 24-hours a day, 365 days a year to respond quickly in the unlikely event that you might need assistance.

Self-charging battery

The Outdoor stairlift always has the optimum amount of power, so you get a smooth ride with a gentle start and stop. It even works if there is a power cut.

Directional controls

Right-handed or left-handed, we can put the joystick controls on either arm, whichever you prefer.

Manual swivel seat

Using the levers enables you to swivel the seat easily, making it simple to get off the chair.

Folding seat, footrest and arms

To store the stairlift away, you can fold away the seat, arm and footrest, keeping the stairway clear for other users.

Safety edges

If there is something blocking the chair on the stairs, the safety edges recognise this and the lift will stop automatically.

Optional call station

You can choose to have a call station to enable you to call the stairlift for yourself, or send it to someone else.

Protective cover

Use the protective cover to keep your stairlift clean and dry when not in use.

Keyswitch


The stairlift can be immobilised when you're not using it.

Seatbelt

Fasten the seatbelt for added security.

Straight rails

Fitted to the stairs and not to the wall, the rail can only be fitted to solid concrete, solid wood or metal staircases.

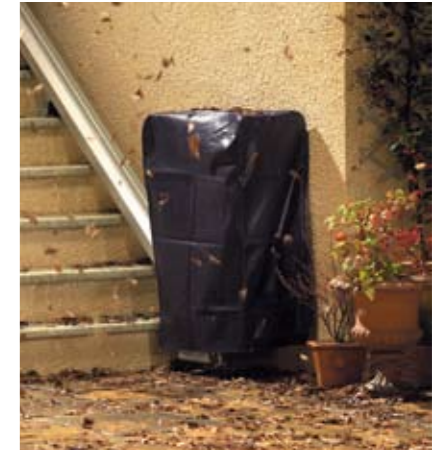



1. Using the Outdoor stairlift is simple and easy. To unfold it, pull down the footrest, the seat and the arms.



2. Once unfolded, make yourself comfortable in the seat. Fasten the seatbelt for extra security.



3. By pressing gently on the controls in the direction you wish to, you can travel up and down the stairs.



4. Once at the top of the stairs, you can swivel the seat to face the landing, allowing you to get off the chair in complete safety.



5. If you wish, you can leave the chair in its swivelled position, creating an effective barrier.



6. When you are not using the stairlift, it can be folded away so that it takes up less space and leaves the stairs clear for other users. Use the protective cover to keep your stairlift clean and dry.

Stannah contact details:
Watt Close, East Portway,
Andover, Hampshire
SP10 3SD

Tel: 0800 715358
Fax: 01264 338043
stannahstairlifts.co.uk

Our company policy is one of continuing product improvement. We therefore reserve the right to alter the product specifications and to change and improve design at any time. Reconditioned products may have different specifications than those illustrated depending on availability. Full operating instructions are supplied for all products. Stannah is a registered trademark. The CE mark indicates that this product complies with the applicable European directives which relate to health, safety, environment and consumer protection. Stannah Stairlifts Limited has been accredited to ISO9001:2008 since 2009. An example of this product has been independently tested and approved. As with any other lift on whose reliability and safety you rely, we recommend that your Stannah stairlift is serviced annually by a trained engineer.



- **Full warranty and service package**
- **24 hour, 365 day call out**
- **Nationwide, locally-based engineers**



The Stannah Promise

**Always true
to our word**

Stannah